




OPERATING INSTRUCTIONS

IONTO-AQUASONIC

Article no. 8826.8345

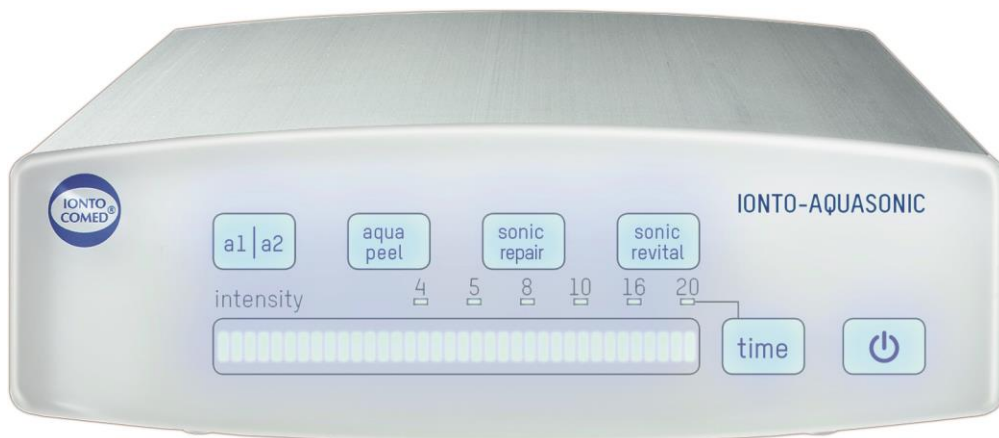

© IONTO Health & Beauty 2018

IONTO Health & Beauty GmbH
Greschbachstr. 3
D-76229 Karlsruhe
Phone +49 721 9770-0
Fax +49 721 9770-290
Email: info@ionto.de
www.ionto.de

Contents

Contents	3
General safety instructions	5
Further Notes	6
Contraindications	7
Treatment Areas	7
Body Treatments	8
Fields of Use	9
Control Elements and Connections	10
IONTO-AQUASONIC	10
Handpieces	11
Installations	12
Device	12
Setting the device up	12
Connecting the device to the mains supply	12
Handpieces	12
Connecting the Handpieces	12
Operation	13
Device	13
Turning the device on	13
Placing the device in standby mode or turning it off	13
Treatment Settings	13
Selecting the Handpiece	13
Selecting a treatment program	14
Selecting the treatment time	14
Setting the intensity	15
Preparing the AQUApeel treatment	15
Preparing the SONICrepair treatment	15
Preparing the SONICrevital treatment	16
Treatment	17
Starting treatment	17
Changing active Handpiece	17
Changing the treatment program	18
Stopping the treatment	18
Cleaning and Care	19
Device	19
Cleaning the device	19
Handpiece	19
Cleaning and disinfecting the Handpiece	20

Repairs and Maintenance ----- 21

Fault, Finding and Solutions-----22

Transport/storage/operating conditions -----23

Disposal -----23

Scope of Delivery ----- 24

Accessories ----- 24

Technical Data -----25



General safety instructions



INFO

The IONTO-AQUASONIC is hereinafter referred to as the device.

- Reading the operating instructions before putting the device into operation is essential. Read the notes carefully. They contain important information on safety during commissioning, use and maintenance of the device.
- Only the manufacturer or personnel who have been authorized by the manufacturer may open the device, unless otherwise explicitly stated in this manual.
- Check the voltage (V) which is specified on the rating plate. The specified voltage must match the voltage which is provided by your electrical utility. Only connect the device to the mains supply after verifying the voltage.
- Using the device with a damaged power cord is prohibited.
- The power cord must not be jammed or pinched and must not come into contact with hot objects, e.g. a heater or a lamp.
- Turn the device off immediately if you discover that the power cord is damaged. Disconnect it from the mains supply. Contact our customer service department. Don't use the device any more until you have received further information from our customer service department.
- Never pull the power cord from the outlet if your hands are wet. Don't yank on the power cord to remove the plug from the outlet.
- Always pull the power cord before you clean the device, reach into it or change the fuse.
- Disconnect the device from the mains supply immediately if liquid or a foreign object gets into the device! Contact our customer service department. Don't use the device any more until you have received further information from our customer service department.
- Don't use the device in wet rooms. Don't pour liquids over the device or immerse it in liquid.
- The device is not intended for use by children or by persons with limited physical, sensory or mental capabilities or a lack of experience or knowledge.
- Turn all mobile phones which are within the treatment environment during treatment off completely.
- Never use the Handpieces to work in body orifices.

Further Notes

- The operating instructions help to operate the device safely. Therefore, always retain the operating instructions for future use. Store the operating instructions by the device so that they are accessible to all users.
- The product and manual are intended for trained cosmeticians.
- This manual may cover the use of functions which are only available with optional equipment.
- If the device was transported, allow some time for the device to reach the ambient temperature before you start using it.
- Only use original accessories.
- After unpacking the device, compare the contents with the specified scope of delivery to ensure that everything is complete and undamaged. Contact our customer service department in case of doubt.
- If you do not use the device properly or use it for purposes other than those described in the user manual, then this releases the manufacturer from any guarantee and warranty obligations whatsoever.

Contraindications



INFO

If in doubt you should consult a doctor.



INFO

For detailed information, please read the treatment instructions!



INFO

Generally only treat healthy people.

Don't use the IONTO-AQUASONIC on the following areas and in case of the following indications:

- acute fever
- acute inflammations
- acute influence of alcohol or drugs
- cancer, suspicion of malignant tumors (only with a clearance of a doctor)
- circulatory insufficiency
- epilepsy
- implanted insulin pump
- cardiac arrhythmias, heart pace maker, heart diseases , coronary diseases
- blood clotting disorder
- bacterial infections (e.g. furuncle, carbuncle, tuberculosis)
- skin diseases and skin infections (e.g. neurodermatitis, psoriasis, eczemas, rosacea)
- immunodeficiency

Treatment Areas

- cardiac region
- spinal cord and brain
- thyroid
- eyeballs
- breasts
- gonads
- metaphysis of children and youths
- acute thrombophlebitis in the treatment area
- acute herpes simplex in the treatment area

- postoperative scars, initial treatment starting from 3 months after the operation or in consultation with the doctor in charge
- skin injuries in the treatment area (e.g. open wounds, acute skin inflammations)
- metal implants in the area of treatment

Body Treatments



INFO

In doubt you should consult a doctor.

- pregnancy (when treating the belly and lumbar region)
- acute articular rheumatism
- recently made radiotherapy
- recently made spinal surgery
- peripheral vascular disease, e.g. thrombosis, varices, phlebitis
- body areas with heavy circulatory disorders

Fields of Use

The IONTO-AQUASONIC is a low-frequency ultrasonic device for cleansing, deep peeling, deep introducing and revitalizing of the skin.

The low-frequency ultrasound is applicated via two specific Handpieces, one with a small metal reed for smaller surfaces such as the face, and one with a wider metal reed for larger surfaces such as the body.

During the treatment the individual Handpiece is moved across the skin and thereby transmits the low-frequency ultrasound.

The IONTO-AQUASONIC treatment can be combined easily with preparations matching the skin condition.

IONTO Health & Beauty recommends the use of the PROLINE CELLULOSE MASKS of the SYMBIOSIS CONCEPT by IONTO.

When using the treatment program SONICrepair for treating blemished skin we recommend using the PROLINE CELLULOSE Mask Green Tea, for dry skin we recommend using the PROLINE CELLULOSE MASK Hyaluron. When using the treatment program SONICrevital for treating mature skin we recommend PROLINE CELLULOSE MASKS Q10 or BEETOX Age Reverse.

The treatment with the IONTO-AQUASONIC results in a variety of effects:

- dead skin cells, scabs and residues of care products containing silicone are gently and efficiently removed
- the blood and lymph circulation is stimulated
- the metabolism is activated
- the regeneration is promoted
- resorption and diffusion processes are promoted

The IONTO-AQUASONIC is successfully applied at a variety of indications:

- atrophic skin
- dry skin
- wan skin colour
- blemished skin
- acne (after consulting the dermatologist)
- hyperceratosis
- scars
- infiltration of active substances

Control Elements and Connections

IONTO-AQUASONIC

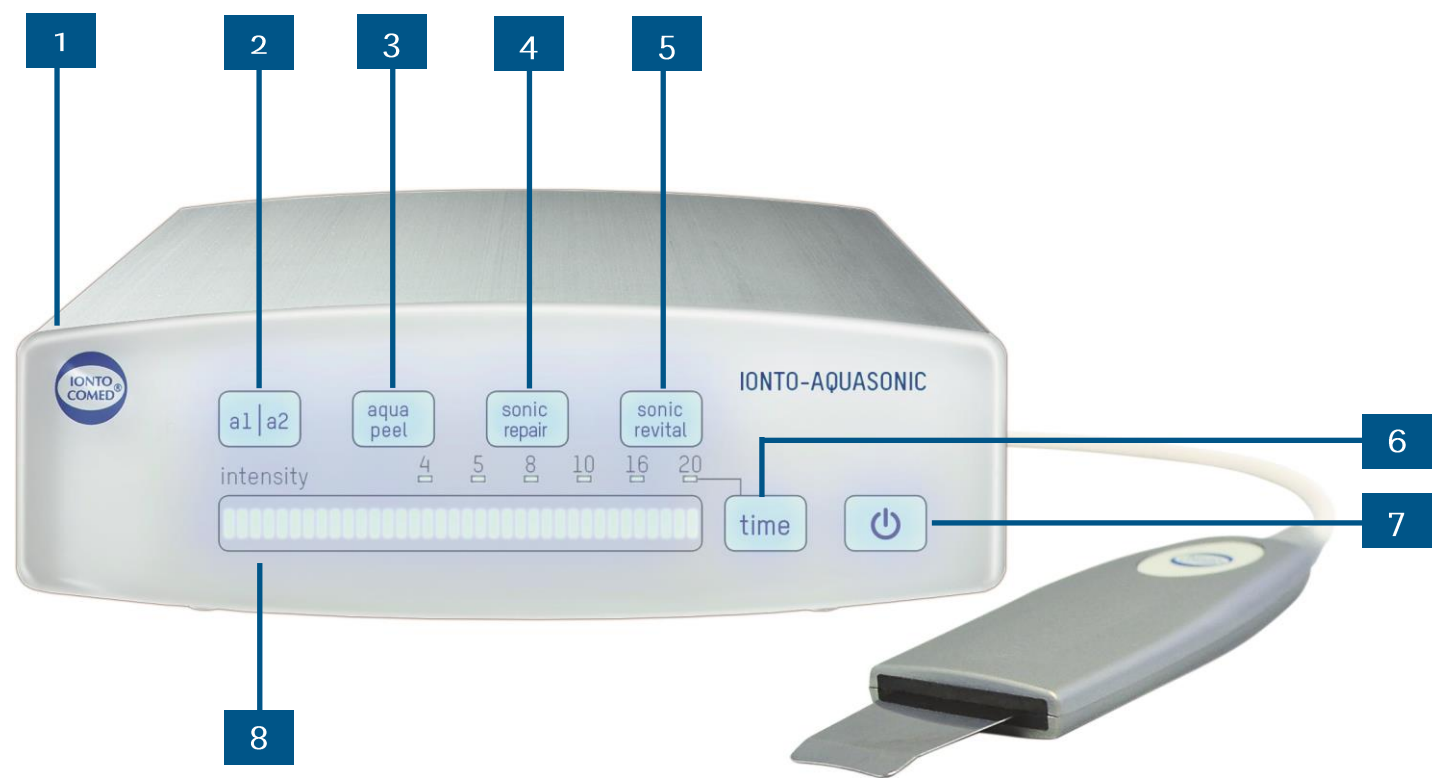


Fig. 1: IONTO-AQUASONIC with Handpiece

1	Casing	2	a1/a2 key (Switch chosen Handpiece)
3	aqua peel key (Program aqua peel)	4	sonic repair key (Program sonic repair)
5	sonic revival key (Program sonic revival)	6	Time key (Set the time)
7	Turn the device on or off / Start or stop treatment	8	Intensity key (Slider with intensity indicator; regulate the intensity)

Handpieces



Fig. 2: Handpieces

9	Handpiece a2	10	Handpiece a1
11	Scoopula on the Handpieces		

Installations

Device

Setting the device up

The place where the IONTO-AQUASONIC is set up must meet certain requirements.

To set the device up correctly:

1. Set the device up on a surface which is level, solid and heat resistant.
2. Don't set the device on a towel or blanket.
3. Ensure that the ventilation slots are free and not covered.

Then the device is situated properly.

Connecting the device to the mains supply

To connect the device to the mains supply:

1. Plug one end of the power cable into the right jack on the back of the device.
2. Plug the other end of the power cable into a power outlet.

Then the device is supplied with power.

Handpieces

Connecting the Handpieces

You can connect 2 Handpieces to the device.

To connect the Handpiece to the device:

- Connect the Handpiece cable to one of the jacks on the back of the device.

The Handpiece is now connected to the device.

Operation

Device

Turning the device on

To turn the device on:

1. Place the mains switch on the back of the device in position <I>.
 - All buttons light up briefly.
 - Then the device is in standby mode.
2. Press and hold the On/Off button **7** for approx. 2 seconds.

Then the device is turned on and the keypad backlighting is active.

Placing the device in standby mode or turning it off

You can turn the device off completely or place it in standby mode.

If the device is in standby mode, it is ready to use more quickly when you want to turn it back on.

To place the device in standby mode or turn it off:

1. To place the device in standby mode, press and hold the On/Off **7** for approx. 2 seconds.
2. Then place the mains switch on the back of the device into position <O> if you want to turn the device off completely.

Then the device is in standby mode or turned off completely.

Treatment Settings



INFO

To provide a good low-frequency ultrasound transmission, make sure that you always apply suitable contact medium to the skin e.g. AQUApeel Lotion (Article no. 9137).

Selecting the Handpiece



INFO

After you have turned on the device, the preset Handpiece is a1.

If you have 2 Handpieces connected to the device, you can choose which Handpiece has to be activated.

To select the Handpiece:

- To switch between the Handpieces a1 and a2, press the button **2**.

Then the selected Handpiece is active.

Selecting a treatment program



INFO

Further information are available in the treatment manual IONTO-AQUASONIC.



INFO

After turning the device on, there is no program pre-selected.

The device offers 3 treatment programs:

► **AQUApeel**

- The deep peeling, combined with the AQUApeel Lotion, is used to remove dead cell layers, scabs, cornifications and residues of care products and makeup.

► **SONICrepair**

- This program subserves the introducing of agents. The skin's own nutrient and oxygen supply is stimulated without stressing the skin.
- We recommend performing the treatment directly on the PROLINE CELLULOSE MASKS Green Tea or Hyaluron.

► **SONICrevital**

- This program subserves the introducing of regenerative agents and anti-aging.
- The circulation and revitalization of the skin is stimulated, the metabolism is accelerated due to the bio stimulation.
- We recommend performing the treatment directly on the PROLINE CELLULOSE MASKS Q10 or BEETOX Age Reverse.

To select a treatment program:

- Press the button with the desired treatment program.

Then the selected treatment program is active.

Selecting the treatment time



INFO

The device offers 6 time settings, 4, 5, 8, 10, 16 and 20 minutes.

To select die treatment time:

- Tap the time button , to switch between the available time settings.

Then the selected treatment time is active.

Setting the intensity

INFO

The intensity scale consists of 36 steps and can be categorized into the areas light, medium and heavy.

To set the intensity:

- To increase the intensity, slide your finger on the surface of the slider **8** to the right.
- To reduce the intensity, slide your finger on the surface of the slider **8** to the left.

Then the set intensity is active.

Preparing the AQUApeel treatment

INFO

Always apply the appropriate contact medium e.g. AQUApeel Lotion (Art. No. 9137) to the skin.

To prepare the AQUApeel treatment:

1. Apply AQUApeel Lotion generously on the treating area as a contact medium.
2. Use the slim flat of the scoopula **11** during the treatment (see Fig. 3).

You can now start the treatment.



Fig. 3: using the scoopula of the Handpiece

Preparing the SONICrepair treatment

To prepare the SONICrepair treatment:

1. Bring on a PROLINE CELLULOSE MASK Green Tea or Hyaluron and treat on the mask or apply an appropriate agent serum.
2. Use the wide flat of the scoopula **11** during the treatment (see Fig. 4).

You can now start the treatment.



Fig. 4: using the scoopula of the Handpiece

Preparing the SONICrevital treatment

To prepare the SONICrevital treatment:

1. Bring on a PROLINE CELLULOSE MASK Q10 or BEETOX Age Reverse and treat on the mask or apply an appropriate agent serum.
2. Use the wide flat of the scoopula **11** during the treatment (see Fig. 4).

You can now start the treatment.

Treatment

Observe the safety instructions to avoid injuries to persons.

Starting treatment



WARNING

RISK OF INJURY!

Don't treat static.

Static treatment can lead to heat pain or inflammatory processes.

- Always keep the Handpiece in motion.
- Immediately cancel the treatment, if uncomfortable feelings occur

To start the treatment:

- To start the treatment, press the On/Off button **7**.
- The On/Off button lights up and the treatment begins.
- You can't change any treatment settings during the treatment.
- When the treatment time has expired, a signal tone sounds up.

The Treatment started.

Changing active Handpiece

If you have connected 2 Handpieces to the device, you can change the active one by interrupting the treatment.



To change the active Handpiece:

1. Press the On/Off button **7** to stop the treatment.
2. Take the other Handpiece.
3. Press the button **2**, to activate the Handpiece.
4. If you want, you can now also change the intensity **8**.
5. To start the treatment, press the On/Off button.

The active Handpiece has changed and the treatment resumes.

Changing the treatment program


To change the treatment program:

1. Press the On/Off button , to stop the treatment.
2. Chose the treatment program by pressing the button for the desired treatment programs.
3. If you want, you can now also change the intensity .
4. To start the treatment, press the On/Off button.

The treatment program has changed and the treatment resumes.

Stopping the treatment

To stop the treatment:

- Press the On/Off button .
- A signal tone sounds up.

The treatment stops.

The remaining time keeps retained.

Cleaning and Care

Please note the following safety notices to avoid mechanical damage and personal injuries.



WARNING

RISK OF ELECTRIC SHOCKS

These can cause deadly injuries.

- Turn off the device before cleaning.
- Disconnect the device from the power supply.

Device

Cleaning the device

NOTE

DAMAGE TO THE DEVICE

Incorrect cleaning can result in damage to the device.

- Don't use any aggressive or abrasive cleaning agents.



INFO

We recommend the disinfectant the Bacillol AF (Art. no. 9220.310).

To clean the device:

1. Clean the device regularly with an aldehyde-based disinfectant.
2. Clean the device housing with a soft sponge cloth.
3. Ensure that no liquid gets into the device housing.

Then the device housing is cleaned.

Handpiece

NOTE

DAMAGE TO THE HANDPIECE

Incorrect cleaning can result in damage to the Handpiece.

Never immerse the Handpiece in liquid.

- Never clean the Handpiece in water, chemicals or in the ultrasonic bath.
- Clean, disinfect and dry the Handpiece after every use.



INFO

We recommend the disinfectant the wipe disinfectant Bacillol Tissues (Article no. 9220.502).

Cleaning and disinfecting the Handpiece

To clean and disinfect the Handpiece:

— Disinfect the metal part of the Handpiece with a wipe disinfectant.

Then the Handpiece is cleaned and disinfected.

Repairs and Maintenance

INFO

The scoopula on the Handpiece wears off over time when used regularly.

We recommend sending in the Handpiece for maintenance annually and changing the scoopula. Contact your distributor.

Apart from that the device doesn't require any maintenance if it is used properly and as intended.

INFO

Repairs may only be performed by authorized specialists from the manufacturer.

Modifications or repairs which were not performed by authorized specialists release the manufacturer from all warranty and liability obligations.

Fault, Finding and Solutions

Fault	Cause	Solutions
The device does not work.	The power cord is not connected properly.	Check the power cord connection. The plugs must be inserted firmly.
	The main switch on the back of the device isn't turned on.	Turn the mains switch on the back of the device on.
	The power outlet is without current.	Check the correct function of the power outlet (e.g. with a table lamp).
The device does not function / the light indicators illuminate.	The On / Off button is not pressed.	Press the On / Off button.

Our customer service is available at +49 721 9770-666 to help you in case of malfunctions or difficulties disassembling or assembling the device.

Transport/storage/operating conditions

Transport and storage temperature:	-20 °C to +50 °C
Operating temperature:	+10 °C to +40 °C
Relative humidity:	max. 90 %, non-condensing

Avoid rapid temperature changes due to the potential for condensation to form in the machine. Only place the device into operation after it has reached the ambient temperature. Only transport the device in the manufacturer's original packaging.

Disposal

Keep the original packaging in case the device needs to be transported in the future. Please dispose of the packaging, the device and accessories in an environmentally friendly manner at corresponding collection sites. All electrical and electronic equipment must be disposed of separately from the general household waste.



If this symbol showing a crossed out garbage can is found on a product, then this product is subject to the European directive 2002/96/EC and must not be mixed with conventional household waste. Separate collection systems exist for these products.

Proper disposal and separate collection of old devices helps to prevent potential damage to the environment and health hazards. It is necessary in order to reuse and recycle used electrical and electronic devices.

Scope of Delivery

Number Article name

1 x	IONTO-AQUASONIC
1 x	AQUASONIC Handpiece a1, large scoopula
1 x	AQUASONIC Handpiece a2, small scoopula
1 x	AQUASONIC Handpiece-holder Duo
1 x	AQUAPEEL Lotion
1 x	PROLINE CELLULOSE MASK, Green Tea

Accessories

Article number Article name

9137	AQUAPEEL Lotion, 250 ml
8228.081	PROLINE CELLULOSE MASK, Green Tea, 10 pieces
8228.095	PROLINE CELLULOSE MASK, Beetox Age Reverse, 10 pieces
8826.201	AQUASONIC Handpiece-holder

Technical Data

Article number:	8826.8345
Power connection	230 oder 110 V 50-60 Hz
Dimensions (W x D x H) in cm:	19 / 17,4 / 6,65
Weight:	Device: 1,0 kg; Device with 2 Handpieces and power adapter: 1,7 kg

IONTO Health & Beauty GmbH
Greschbachstr. 3 • D-76229 Karlsruhe
Phone: +49 721 9770-0 • Fax: +49 721 9770-290
www.ionto.de • Email: info@ionto.de

Printed in Germany-8826.8345 -1.2-08/2018