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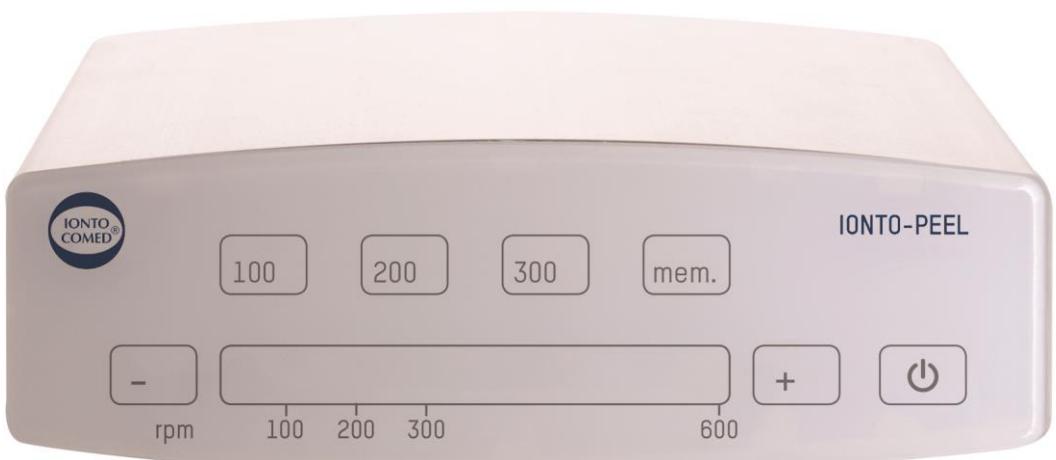
IONTO-PEEL

Art.-Nr. 8826.8312

© IONTO Health & Beauty 2018

IONTO Health & Beauty GmbH
Greschbachstr. 3
D-76229 Karlsruhe
Telefon +49 721 9770-0
Telefax +49 721 9770-290
Email: info@ionto.de
www.ionto.de

Inhalt

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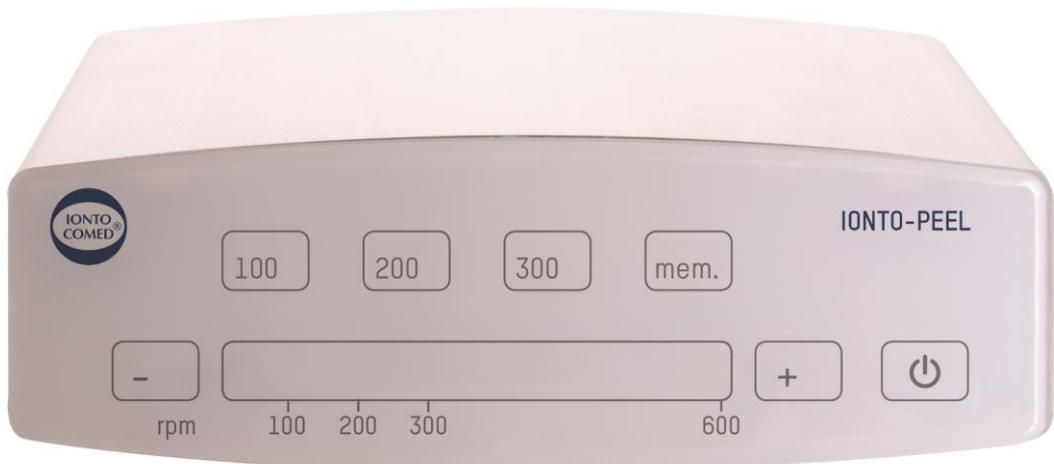
Contents

Contents -----	3
Safety Instructions -----	5
Contraindication -----	6
Notes for Working -----	7
Fields of Use -----	8
Control Elements and Connections -----	9
IONTO-PEEL -----	9
PEEL-Handpiece -----	10
Installations -----	11
IONTO-PEEL -----	11
Setting the IONTO-PEEL-----	11
Connecting the IONTO-PEEL to the mains supply -----	11
PEEL Handpiece -----	12
Connect the PEEL Handpiece -----	12
Fitting attachment to the PEEL Handpiece -----	12
Removing the attachment from the PEEL Handpiece-----	13
Operation -----	14
IONTO-PEEL -----	14
Switch the IONTO-PEEL ON -----	14
Placing the IONTO-PEEL in standby mode or turning it OFF -----	14
Treatment-----	14
Treatment settings-----	14
Adjusting the speed-----	14
Starting treatment -----	15
Ending the treatment -----	15
Memory function-----	15
Saving the speed setting -----	15
Repairs and Maintenance -----	17
Cleaning and Care-----	18
IONTO-PEEL -----	18
Cleaning the IONTO-PEEL -----	18
PEEL Handpiece -----	18
Cleaning the PEEL Handpiece -----	18
Brushes, pumice stones, sponge holders and sponges -----	19
Cleaning the brushes, pumice stones, sponge holders and sponges -----	19

Contents

Fault, Finding and Solutions-----	20
Transport/storage/operating conditions -----	21
Disposal -----	21
Scope of delivery-----	22
Accessories -----	22
Technical Data -----	23

Safety Instructions

- Reading the operating instructions before putting the IONTO-PEEL into operation is essential. Read the notes carefully. They contain important information on safety during commissioning, use and maintenance of the IONTO-PEEL.
- Only the manufacturer or personnel who have been authorized by the manufacturer may open the IONTO-PEEL, unless otherwise explicitly stated in this manual.
- Check the voltage (V) which is specified on the rating plate. The specified voltage must match the voltage which is provided by your electrical utility. Only connect the IONTO-PEEL to the mains supply after verifying the voltage.
- Using the IONTO-PEEL with a damaged power cord is prohibited.
- The power cord must not be jammed or pinched and must not come into contact with hot objects, e.g. a heater or a lamp.
- Turn the IONTO-PEEL off immediately if you discover that the power cord is damaged. Disconnect it from the mains supply. Contact our customer service department. Don't use the IONTO-PEEL any more until you have received further information from our customer service department.
- Never pull the power cord from the outlet if your hands are wet. Don't yank on the power cord to remove the plug from the outlet.
- Always pull the power cord before you clean the IONTO-PEEL, reach into it or change the fuse.
- Disconnect the IONTO-PEEL from the mains supply immediately if liquid or a foreign object gets into the IONTO-PEEL! Contact our customer service department. Don't use the IONTO-PEEL any more until you have received further information from our customer service department.
- Don't use the IONTO-PEEL in wet rooms. Don't pour liquids over the IONTO-PEEL or immerse it in liquid.
- The IONTO-PEEL is not intended for use by children or by persons with limited physical, sensory or mental capabilities or a lack of experience or knowledge.
- Turn all mobile phones which are within the treatment environment during treatment off completely.

Contraindication

INFO

In case of doubt you should consult with a physician.

INFO

In general, only treat healthy people.

Don't use the IONTO-PEEL on the following areas and in case of the following indications:

- Inflamed areas of skin
- Herpes Simplex

Notes for Working

- The operating instructions help to operate the IONTO-PEEL safely. Therefore, always retain the operating instructions for future use. Store the operating instructions by the IONTO-PEEL so that they are accessible to all users.
- The product and manual are intended for trained cosmeticians.
- This manual may cover the use of functions which are only available with optional equipment.
- If the IONTO-PEEL was transported, allow some time for the IONTO-PEEL to reach the ambient temperature before you start using it.
- Only use original accessories.
- After unpacking the IONTO-PEEL, compare the contents with the specified scope of delivery to ensure that everything is complete and undamaged. Contact our customer service department in case of doubt.
- If there are any ventilation slots, be sure not to cover these. Don't cover the IONTO-PEEL and don't install it inside compartments/drawers.
- If you do not use the IONTO-PEEL properly or use it for purposes other than those described in the user manual, then this releases the manufacturer from any guarantee and warranty obligations whatsoever.

Fields of Use

The IONTO-PEEL offers speeds of 100 to 300 RPM.

3 speeds can be selected directly at the push of a button.

Another custom speed can also be saved to a separate button.

Control Elements and Connections

IONTO-PEEL

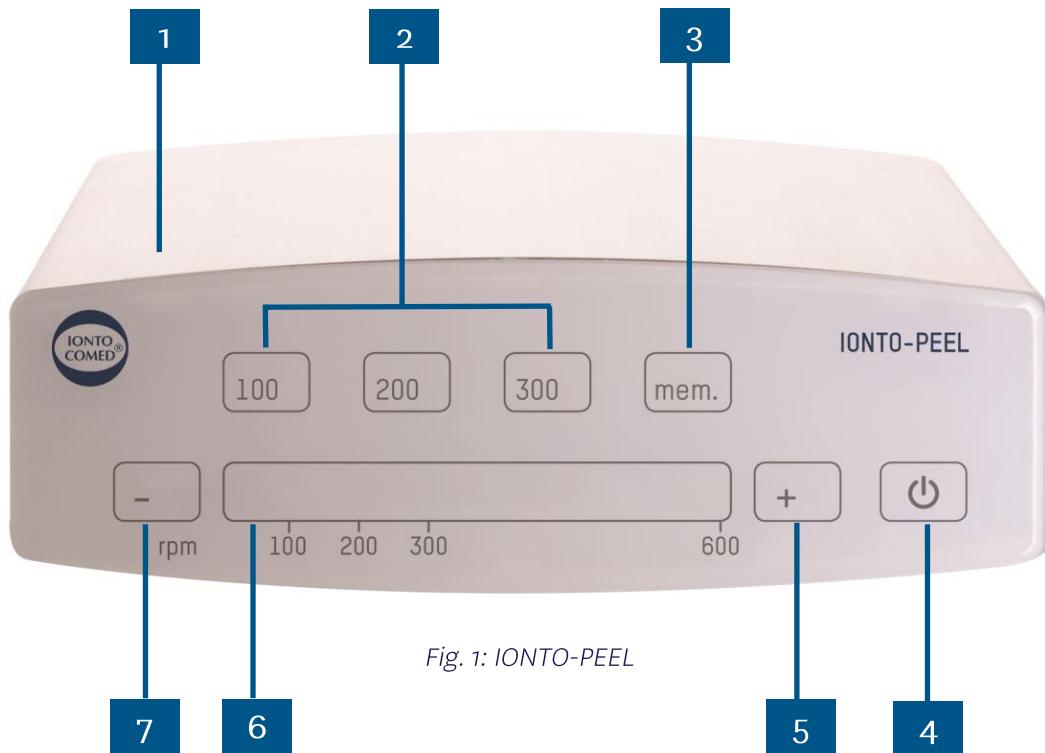
Fig. 1: IONTO-PEEL

1	Housing	2	Quick select for 100, 200 or 300 RPM
3	Memory button	4	switch the IONTO-PEEL ON or OFF / start or stop the treatment
5	Increase the speed	6	Slider with speed indicator
7	Reduce the speed		

PEEL-Handpiece

Fig. 2: PEEL-Handpiece

8 replaceable attachment

9 Attachment holder

10 Handpiece handle

Installations

IONTO-PEEL

Setting the IONTO-PEEL

The place where the IONTO-PEEL is set up must meet certain requirements.

To set the IONTO-PEEL up correctly:

1. Set the IONTO-PEEL up on a surface which is level, solid and heat resistant.
2. Don't set the IONTO-PEEL on a towel or blanket.
3. Ensure that the ventilation slots are free and not covered.
4. If you own several IONTO-COMED® Essential Line IONTO-PEELs, you can stack up to 3 IONTO-PEELs on top of one another.

Then the IONTO-PEEL is situated properly.

Connecting the IONTO-PEEL to the mains supply

To connect the IONTO-PEEL to the mains supply:

1. Plug one end of the power cable into the right jack on the back of the IONTO-PEEL.
2. Plug the other end of the power cable into a power outlet.

Then the IONTO-PEEL is supplied with power.

Fig. 3: Backside of the IONTO-PEEL

PEEL Handpiece

Connect the PEEL Handpiece

To connect the PEEL Handpiece to the IONTO-PEEL:

- Connect the PEEL Handpiece cable to the left jack on the back of the IONTO-PEEL.

The PEEL Handpiece is now connected to the IONTO-PEEL.

Fig. 4: Connection for the PEEL-Handpiece

Fitting attachment to the PEEL Handpiece

 **INFO** Attachments include brushes, pumice stones or sponge holders.

Only IONTO-COMED attachments fit the thread of the PEEL Handpiece.

To fit the attachment to the PEEL Handpiece:

1. Ensure that the Handpiece isn't switched ON.
2. Hold the attachment holder **9** securely.
3. Screw the attachment onto the thread in the clockwise direction, ensuring that it goes on straight.

Then the attachment is fitted to the PEEL Handpiece.

Fig. 5: to fit the Attachment

Removing the attachment from the PEEL Handpiece

To remove the attachment from the PEEL Handpiece:

1. Ensure that the Handpiece isn't switched ON.
2. Hold the attachment holder **9** securely.
3. Unscrew the attachment from the thread in the counterclockwise direction.

The attachment is now removed from the PEEL Handpiece.

Now you can fit another attachment to the PEEL Handpiece.

Operation

IONTO-PEEL

Switch the IONTO-PEEL ON

To switch the IONTO-PEEL ON:

1. Place the mains switch on the back of the IONTO-PEEL in position <|>. — Then the IONTO-PEEL is in standby mode.
2. Press and hold the ON/OFF button **4** for 2 seconds.

Then the IONTO-PEEL is switched ON and the keypad backlighting is active.

Placing the IONTO-PEEL in standby mode or turning it OFF

You can switch the IONTO-PEEL OFF completely or place it in standby mode.

If the IONTO-PEEL is in standby mode, it is ready to use more quickly when you want to switch it back ON.

To place the IONTO-PEEL in standby mode or switch it OFF:

- To place the IONTO-PEEL in standby mode, press and hold the ON/OFF button **4** for 2 seconds.
- Then place the mains switch ON the back of the IONTO-PEEL into position <O> if you want to switch the IONTO-PEEL OFF completely.

Then the IONTO-PEEL is in standby mode or switched OFF completely.

Treatment

Treatment settings

Adjusting the speed

The IONTO-PEEL operates at speeds between 100 and 300 RPM.

You can select the predefined speeds at the push of a button or adjust the speed with the (+) and (-) buttons or the slider.

To adjust the speed:

1. To set the speed to 100, 200 or 300 RPM directly, press the corresponding button **2**.

2. To increase the speed incrementally, press the (+) button **5** or slide your finger to the right on the slider surface **6**.

3. To decrease the speed incrementally, press the (-) button **7** or slide your finger to the left on the slider surface **6**.

The speed is adjusted.

Starting treatment

The IONTO-PEEL must be switched ON in order to start the treatment.

To start the treatment:

— Press the ON/OFF button **4**.

The treatment starts.

Ending the treatment

To end the treatment:

— Press the ON/OFF button **4**.

The treatment stops.

The settings are retained.

Memory function

Saving the speed setting

To save the current rotational speed setting:

1. Use the (+) **5** and (-) buttons **7** or the slider **6** to set the rotational speed you want to save.
2. Press and hold the memory button **3** for 3 seconds.

— The signal only switches OFF after you release the memory button.

This saves the rotational speed setting to the memory button.

Loading the speed setting

To load the saved rotational speed setting:

- Press the memory button  .

A beep sounds and the rotational speed setting which was saved to the memory button is set.

Repairs and Maintenance

INFO

The IONTO-PEEL doesn't require any maintenance if it is used properly and as intended.

Repairs may only be performed by authorized specialists from the manufacturer.

Modifications or repairs which were not performed by authorized specialists release the manufacturer from all warranty and liability obligations.

Cleaning and Care

Please note the following safety notices to avoid mechanical damage and personal injuries.

RISK OF ELECTRIC SHOCK!

These can lead to fatal injuries.

- Switch OFF the IONTO-PEEL before cleaning it.
- Always disconnect from the mains power supply before cleaning.

IONTO-PEEL

Cleaning the IONTO-PEEL

NOTE

DAMAGE TO THE IONTO-PEEL

Incorrect cleaning and care can result in damage to the IONTO-PEEL. This causes housing damage.

- Do not use any aggressive or abrasive cleaning agents.

To clean the IONTO-PEEL:

1. Clean the IONTO-PEEL regularly with an aldehyde-based disinfectant.
2. Clean the IONTO-PEEL housing with a soft sponge cloth.
3. Ensure that no liquid gets into the IONTO-PEEL housing.

Then the IONTO-PEEL housing is cleaned.

PEEL Handpiece

Cleaning the PEEL Handpiece

NOTE

DAMAGE TO THE PEEL HANDPIECE

Incorrect cleaning and care can result in damage to the IONTO-PEEL.

- Do not use any aggressive or abrasive cleaning agents.
- Never immerse the PEEL Handpiece in liquid.
- Never clean the PEEL Handpiece with water, chemicals or in an ultrasonic bath.

To clean the PEEL Handpiece:

- Wipe the PEEL Handpiece down with disinfectant.
- Ensure that no liquid gets inside the Handpiece.

This completes cleaning of the PEEL Handpiece.

Brushes, pumice stones, sponge holders and sponges

NOTE

DAMAGE TO THE PUMPIC STONES

Incorrect cleaning and care can result in damage to pumice stones

- Never clean the pumice stones an ultrasonic bath.

 INFO

Disinfect, clean and dry brushes, pumice stones, sponge holders and sponges after each use.

Request our free sanitation plan.

 INFO

We recommend the combined cleaning agent and disinfectant Gigasept Instru AF (item no. 9220.744).

Cleaning the brushes, pumice stones, sponge holders and sponges

To clean brushes, pumice stones, sponge holders and sponges:

1. Remove product residue under running water.
2. Place the parts in a disinfection pan or an ultrasonic bath.
3. Alternately, disinfect the parts via spray disinfection.
4. Then wash the parts with a disinfectant soap.
5. Dry the parts.

Then the brushes, pumice stones, sponge holders and sponges are cleaned.

Fault, Finding and Solutions

Fault	Cause	Solution
The IONTO-PEEL does not work.	The power cord is not connected properly.	Check the power cord connection. The plugs must be inserted firmly.
	The main switch on the back of the IONTO-PEEL isn't switched ON.	Switch the mains switch on the back of the IONTO-PEEL ON.

In the event of any other problem arising which is not dealt with above, please contact our customer service department (telephone number +49 721-9770-666).

Transport/storage/operating conditions

Transport and storage temperature:

-20 °C to +50 °C

Operating temperature:

+10 °C to +40 °C

Relative humidity:

max. 90%, non-condensing

Avoid rapid temperature changes due to the potential for condensation to form in the machine.
Only place the device into operation after it has reached the ambient temperature.

Only transport the device in the manufacturer's original packaging.

Disposal

Keep the original packaging in case the device needs to be transported in the future.

Please dispose of the packaging, the device and accessories in an environmentally friendly manner at corresponding collection sites. Dispose of used micro abrasion crystals and used air filters according to the sanitation regulations.

All electrical and electronic equipment must be disposed of separately from the general household waste.

If this symbol showing a crossed out garbage can is found on a product, then this product is subject to the European directive 2002/96/EC and must not be mixed with conventional household waste. Separate collection systems exist for these products.

Proper disposal and separate collection of old devices helps to prevent potential damage to the environment and health hazards. It is necessary in order to reuse and recycle used electrical and electronic devices.

Scope of delivery

Number Article name

1 x	IONTO-PEEL
1 x	PEEL Handpiece

Accessories

Article number Item name

60139	PEEL Handpiece
8826.1	Universal Handpiece holder
8257	SilkySky Ø 45 mm, for the face
8360	Pumice stone, Ø 20 mm, for small areas of skin
8260	Pumice stone, Ø 45 mm, for the face and body
8361	Sponge holder, reusable
8357	BlueBay, Ø 45 mm, for tough body skin

Technical Data

Article number:	8826.8312
Voltage:	230 / 110 Volt; 50 – 60 Hz
Power consumption:	13 VA
Protection class:	II
Dimensions (W x D x H) in mm:	190 / 173 / 61
Weight:	1.5 kg

IONTO Health & Beauty GmbH
Greschbachstr. 3 • D-76229 Karlsruhe
Telefon: +49 721 9770-0 • Telefax: +49 721 9770-290
<http://www.ionto.de> • Email: info@ionto.de

Printed in Germany-8826.8312-1.2-07/2018